

DOWNLOADING AND INSTALLING THE ACCUFIT 9000 SOFTWARE & UART DRIVER (CP210X)




Downloading and Installing the AccuFIT 9000 Software and UART Driver (CP210x)


1. First determine whether your operating system is **64-bit** or **32-bit** version.

To determine this, follow the instructions below:


- a. On the Taskbar Click on the **Start** button and choose **Settings**.

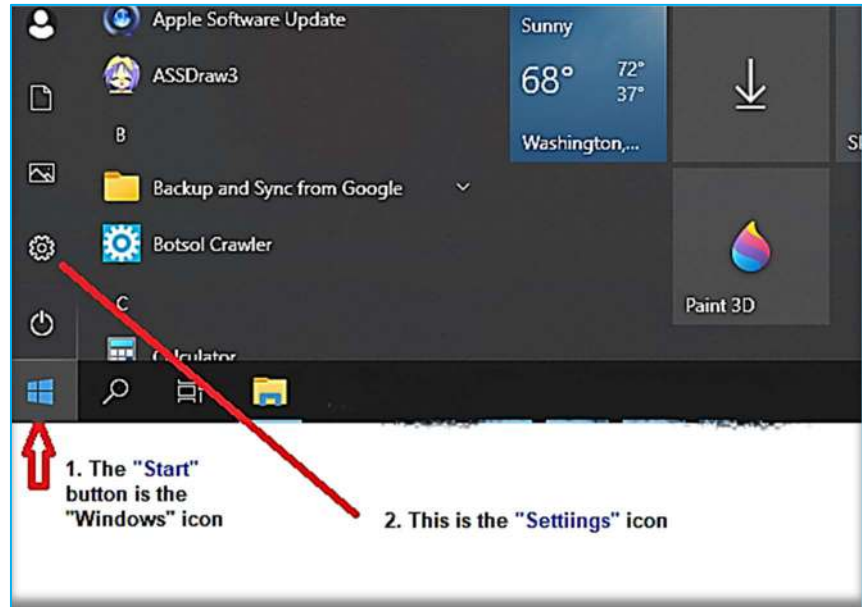


- b. After choosing **Settings**, Click on **System** (as shown highlighted with the red-bordered box in the image on the right)

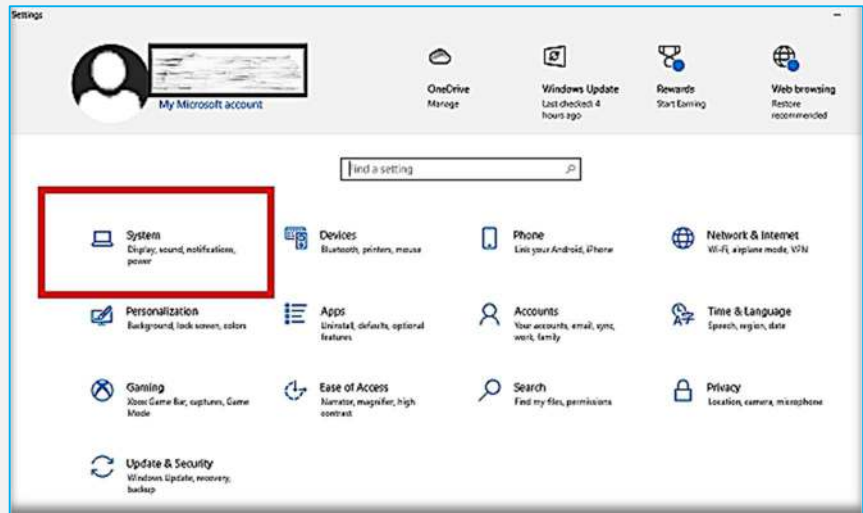


c. In **System** menu, choose **About**.
(shown highlighted with the red-bordered box in the image on the right)
Note: You will have to cursor down to the bottom of the list to see **About**.

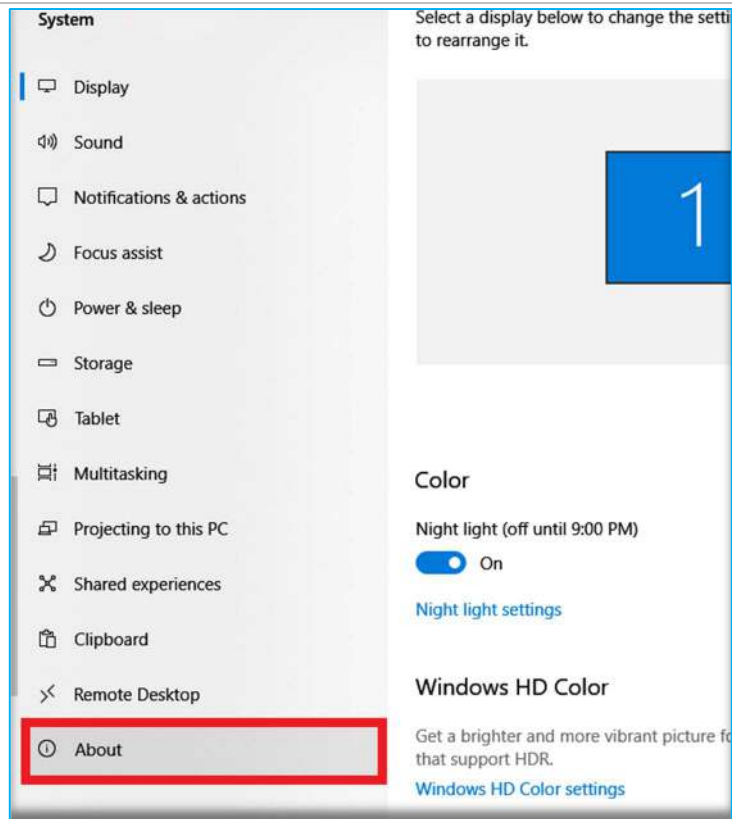

d. The bit-version (32-bit or 64-bit) is displayed in the System type field.
(shown highlighted with the red-bordered box in the image on the right)

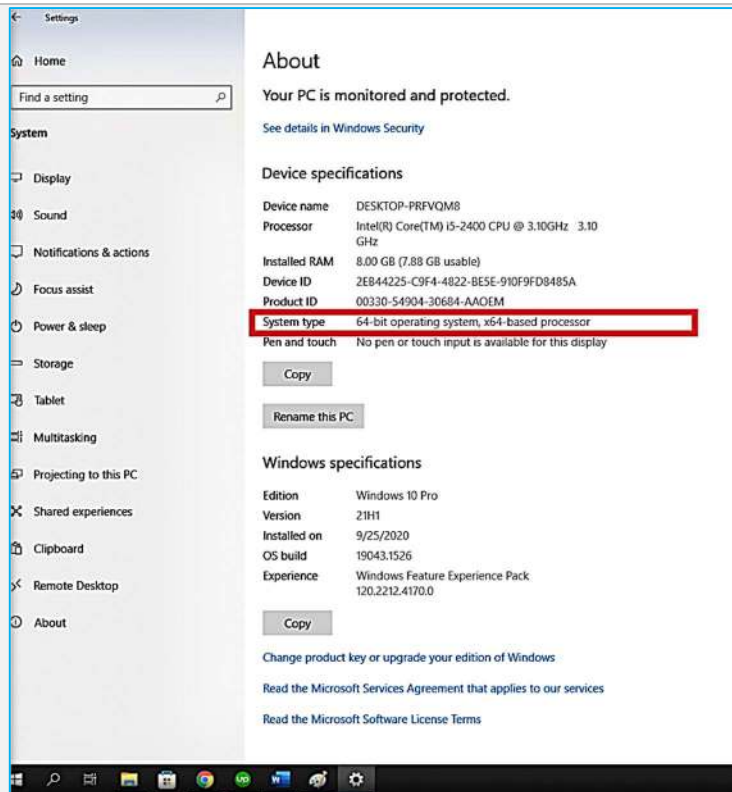


2. Point your browser to the AccuTec website at <https://accutec.com/support/> where you will see the downloads for the AccuFIT software and Driver.

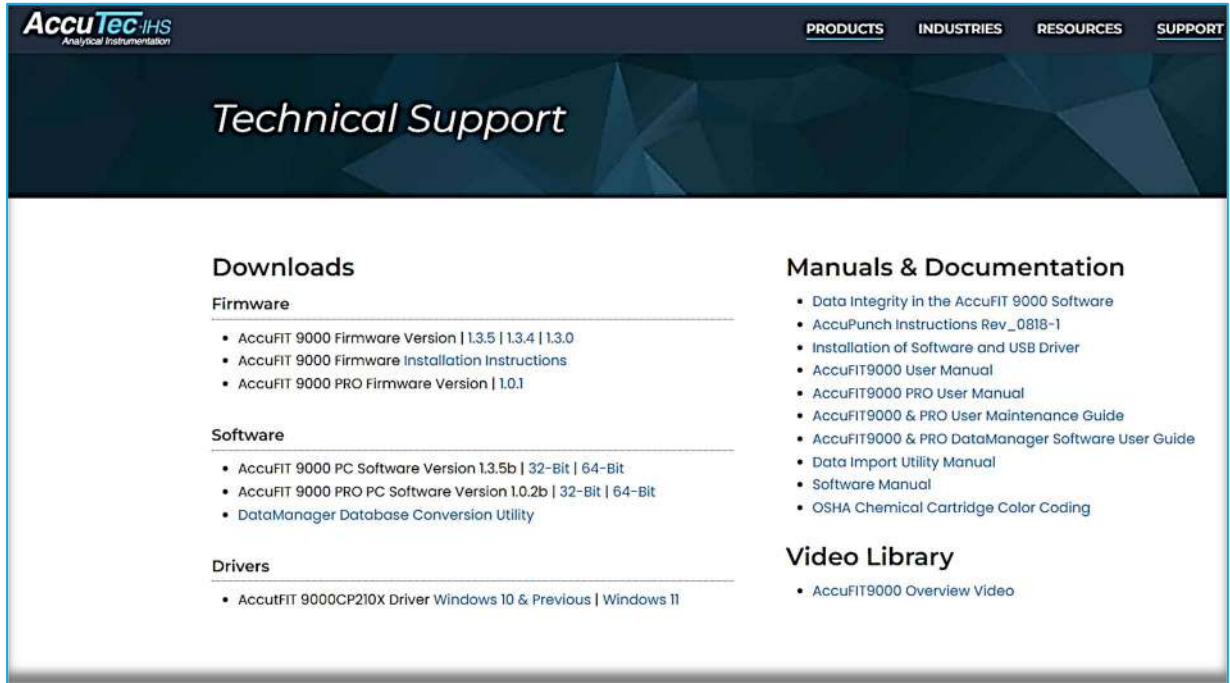

3. Select the appropriate Software (AccuFIT 9000 or AccuFIT 9000 PRO) for 64-bit or 32-bit processor/operating system depending on your computer's configuration. Download this file by clicking on the blue hypertext.


4. Next download the CP210x UART (universal asynchronous receiver/transmitter) driver.

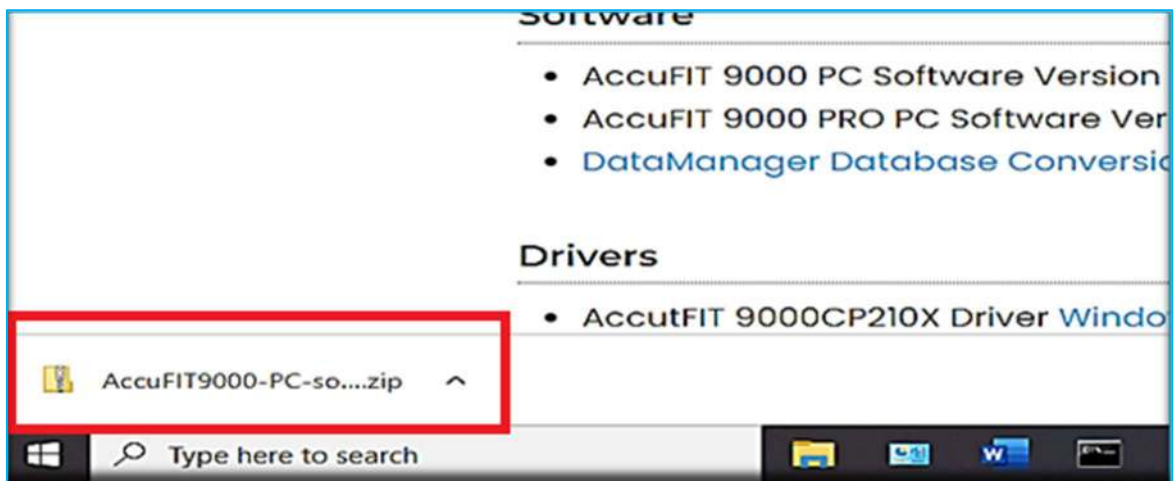


- Right-click on the zipped file shown above and select "Show in Folder"

- This will now show your Download Folder which will have as the first two items the downloaded AccuFIT Software.

Note: the example below shows only the CP210x driver, but if you have downloaded both the driver and the software, both items will be present as “zipped” files.

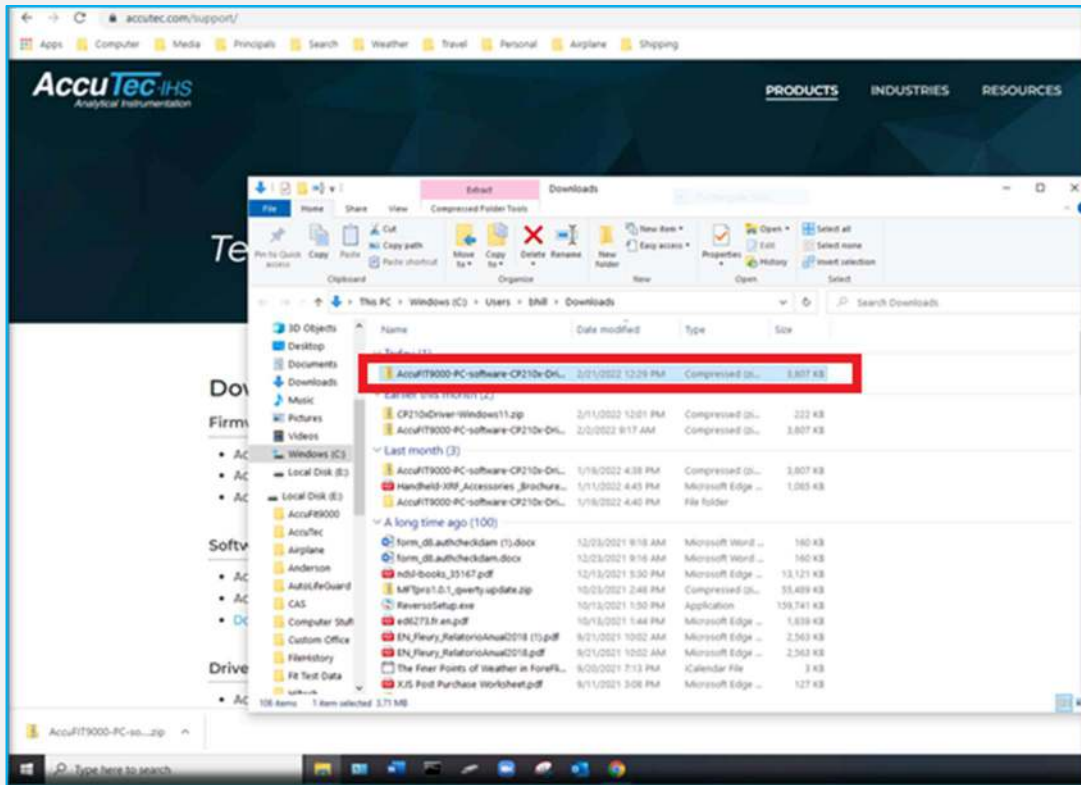

- Click on the downloaded file you wish to install (*in this case we will use the CP210x driver file as the example*). This will open an “Extract” function (shown in pink at the top of the window), and a list of several files will appear.

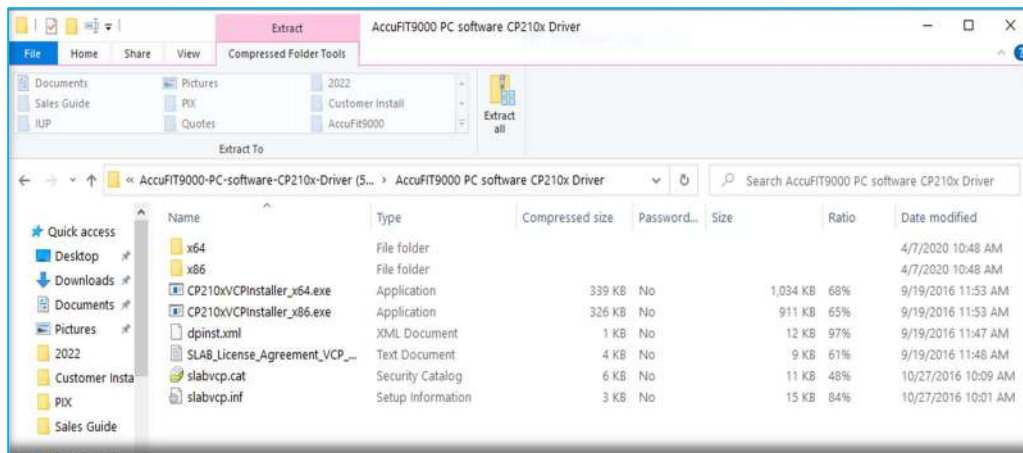


- c. Begin the installation by clicking on either:
- “CP210xVCPInstaller_x64.exe” for the 64-bit driver or
 - “CP210xVCPInstaller_x86.exe” for the 32-bit driver, and follow prompts.

NOTE: A blue Microsoft Defender Smart Screen warning (see image below) **may** appear telling you that this installation has been blocked.

If it does, Click on “**More info**” and choose “**Run Anyway**”. Then follow prompts to complete installation.




6. Next, in your Downloads Folder, click on your Software download. This will bring up an “Extract” window as shown below (note: in this example we are loading the AccuFIT 9000 PRO Software for 64-bit computers):

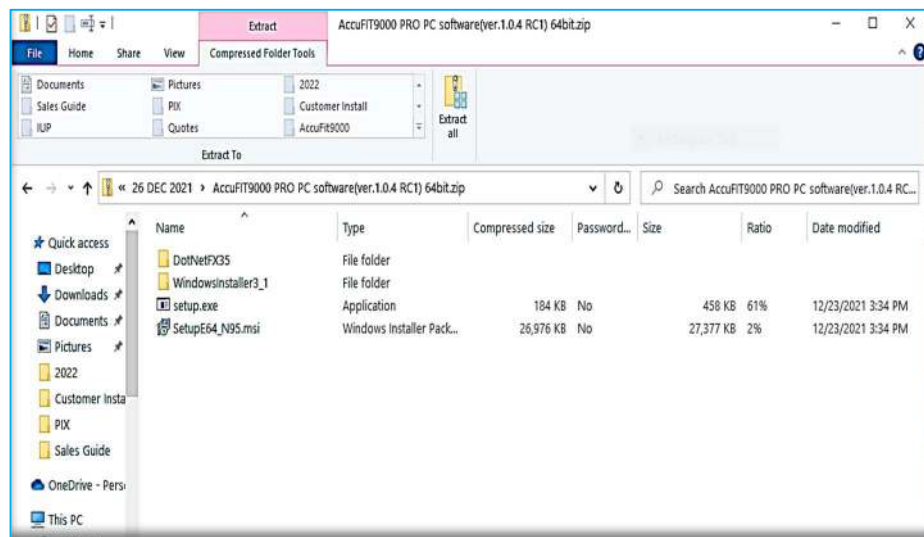


7. Click on **“SetupE64_N95.msi”** and follow the prompts to install. You will probably see the blue Windows Defender Smart Screen, but click on **“More info”** and **“Run anyway”** to continue.
8. On the Taskbar in the box where **“Type here to search”** appears (image A), enter **“Control Panel”** and see an image as shown in image B.

A)

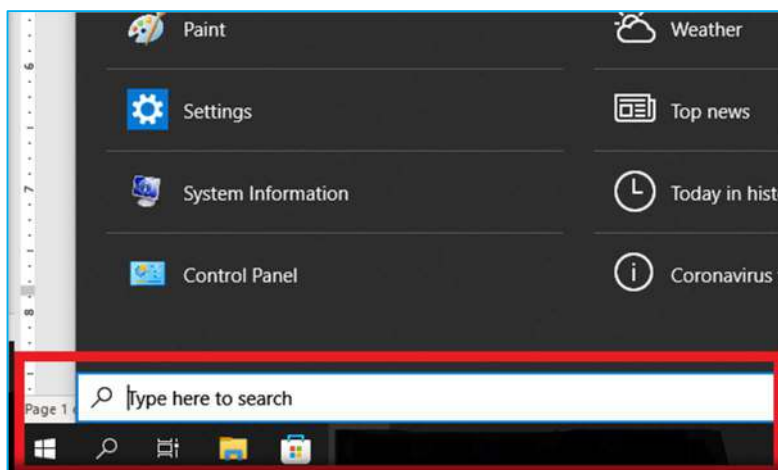


Type in
**“Control
Panel”** here.

B)

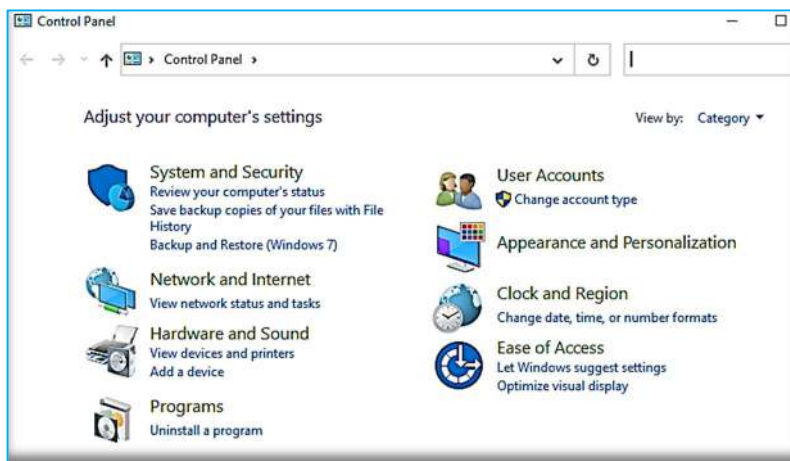


- Open the Control Panel and click on “**Programs** and then on “**Programs and Features**”

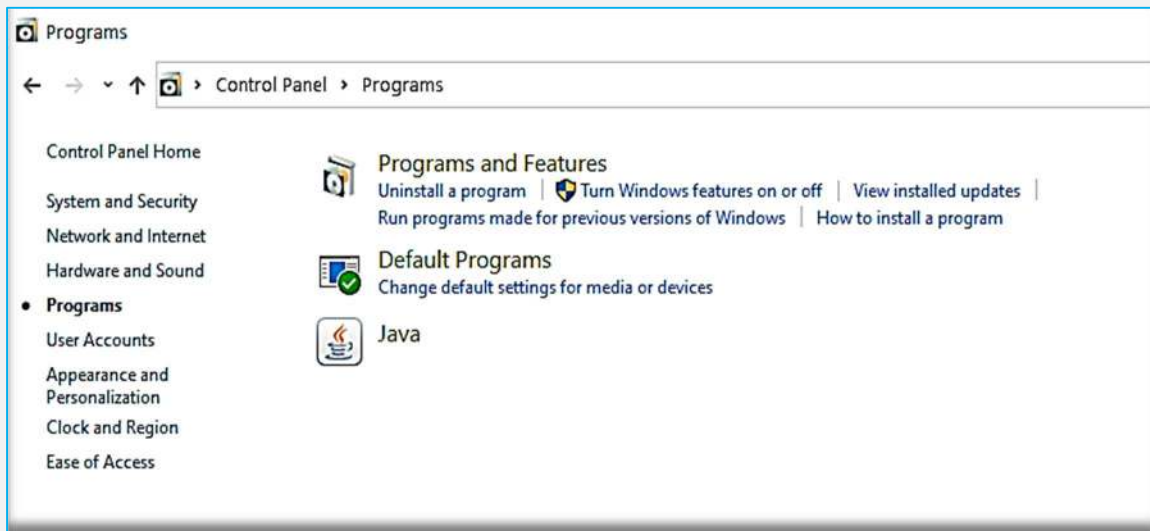


- This will bring up an alphabetical list of installed programs. **The AccuFIT 9000** software should be one of the first listed.

Next scroll down to see “**Windows Driver Package- Silicon Laboratories, Inc**”

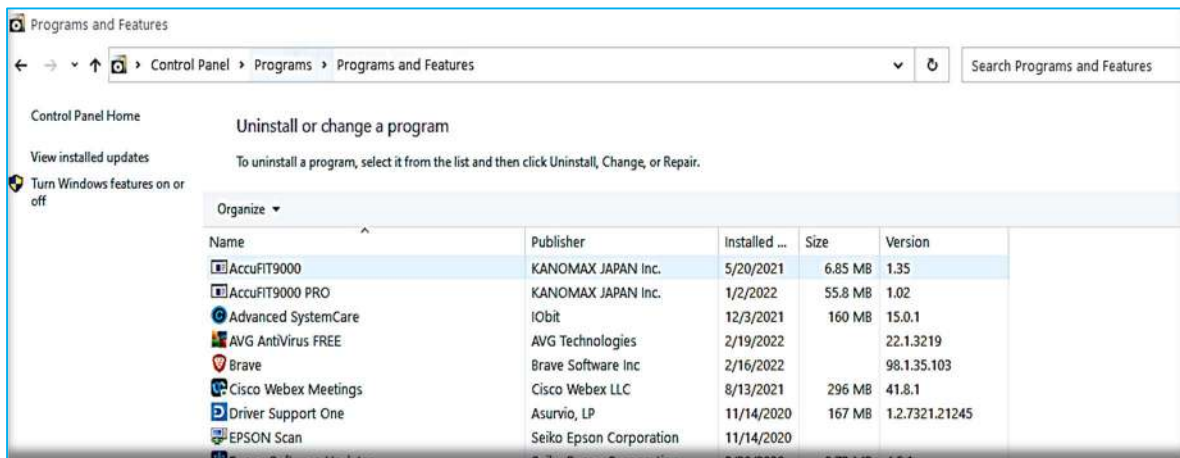

- If both of these programs are shown in your Program List, the software and driver have been successfully installed.

12. In the upper left corner of the Program List in Control Panel (shown in the previous image) click on “Turn Windows Features on or off”. The following window will open:

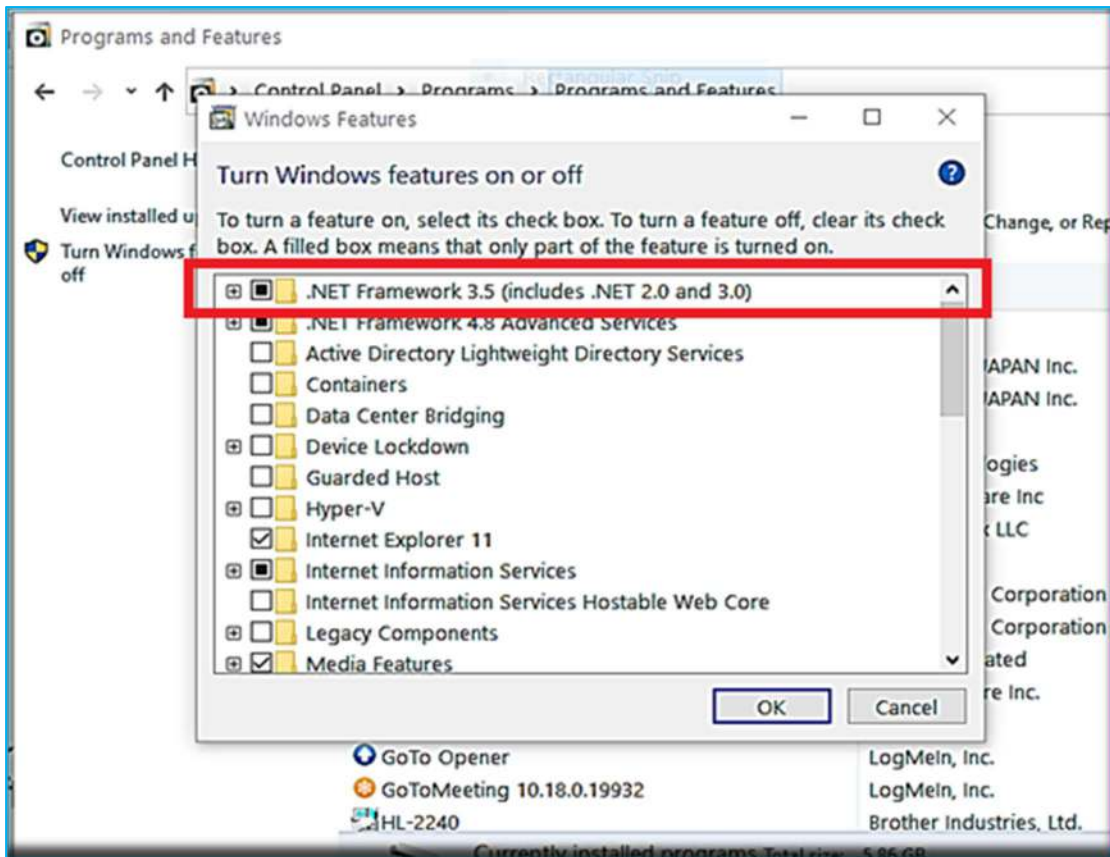

Make sure that .NET Framework 3.5 has a black dot in the left-hand box.

13. Next click on the **“Start”** menu containing the Program list.

Under the **“A”** group, AccuFIT 9000 or AccuFIT 9000 PRO should appear.

Click on the entry and two subfolders will appear.

Place your cursor on the **“AccuFIT 9000 Software”** (or **“AccuFIT 9000 PRO Software”**). Hold down the left mouse button, and drag the file to your desktop. This will create an icon on the Desktop which will be easier to access to start the application.

